



Surgical Hand Brush with Chlorhexidine

Disposable

Surgical hand brushes are used to significantly reduce the number of micro-organisms on the hands and forearms before performing surgeries. The sponge on the back of the brush is used for dipping in and sucking disinfecting fluid for cleaning parts rubbed. The nail cleaner feature makes the removal of micro-organisms under the nail quick and easy.

Intended Use

Used to Hospital and clinic nursing.

Specifications

·Item	Surgical Hand Brush with Povidone iodine
·Spec.	Sponge Brush 80*50*25mm(±10%) Povidone iodine 7.5% / 10% Thoracic Fluid Content 20ml
·Material	Foam sponge with plastic holder Soft brush body made of LDPE
·Content	Foam,nail cleaner,surgical brush
·Function	Disposable
·Sterilization	EO Gas



Caution

- Not used to disinfect invasive medical devices.
- Do not use it when the package is unpacked or damaged.
- In the event of skin irritation or rash, discontinuance
- Avoid contact with eyes, and keep away from children.





Specifications Details



Item No	Item	Description	Sterile
E20037	Dry Surgical Scrub Brush	Sponge Brush 8x5x2.1cm	300pcs/ctn 66x28x49cm 6/5kgs
E20038	Surgical Hand Brush with Iodine	Sponge Brush 8x5x 2.1cm With Iodine 7.5%/10% 20ml	300pcs/ctn 66x28x49cm 14.5/13.5kgs
E20039	Surgical Hand Brush with Chlorhexidine	Sponge Brush 8x5x 2.1cm With chlorhexidine gluconate solution 4% 20ml	300pcs/ctn 66x28x49cm 14.5/13.5kgs

Directions

1. Wet hands and forearms with water.
2. Clean under nails with nail cleaner, scrub for 3 minutes with the impregnated brush.
3. Pay close attention to the nails, cuticles, and interdigital spaces.
4. Rinse thoroughly with running water.

Storage and transportation

1. The device shall be stored at a relative humidity of less than 80%, and the storage environment shall be kept non-corrosive gases, dry, avoiding sunlight, well-ventilated conditions, and clean.
2. The device shall be taken care of during transportation and handling.

Packing Details

Package: 30pcs/box, 300pcs/ctn
 Carton size: 66x28x49cm
 G.W/N.W: 14.5/13.5kgs

