



Wound Skin Closure Strip

Disposable

Wound closure tape strips are made of non-woven, rayon-backed material reinforced with filaments for strength coated with Hypoallergenic adhesive, porous surgical tape strips which can be used to close small wounds.

Provides wound support and assists in increasing the tensile strength of the wound. They are applied across the laceration in a manner that pulls the skin on either side of the wound together.

Wound closure strips may be used instead of sutures (stitches) in some injuries because they lessen scarring and are easier to care for.

Usage

Wound closure strips are ideal for use in highly contoured areas or areas of musculoskeletal movements, such as joints. They are also for use in areas where swelling, edema, hematomas, or bloating may occur.



Benefits

Wound closure strips help prevent maceration. They result in less scarring when compared to staples or sutures. They present a lesser chance of infection than sutures or staples.

Features

- It is a good assistant for sewing up the surgical wound caused by surgical and cosmetic surgery operations.
- Caring for the surgical wound after taking out stitches, reducing scar formation.
- Bonding and repairing the little wound by hurt in the daily lift, such as the stab wound, the cut wound, and so on.





1


Step

- First, clean the wound with physiological saline and keep it dry completely.
- Second, paste the closure from the 2 edges successively to avoid the wound dehiscence.
- Third, apply it from the two edges of the wound to the middle for avoiding the wound dehiscence.
- Finally, press the closure smoothly to keep it pasted well, and assure there are no crevices.



2

Illustration for how to use wound skin closure strip

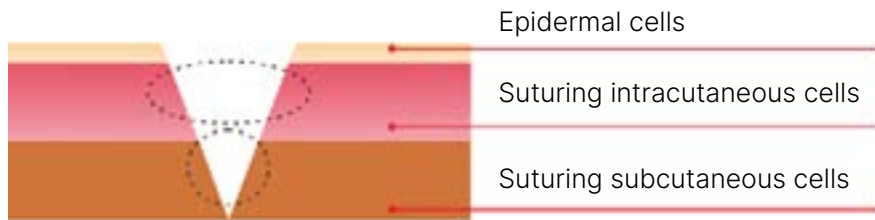

3



After suturing intracutaneous and subcutaneous cells, make the skin flat with wound skin closures strip.

Specification	Application	Pcs/box	Boxes/carton
12mm*100mm (3 Strips)	Wound of 50-100mm long	50pcs/box	40boxes/carton
6mm*100mm (5 Strips)	Wound of 50-100mm long	50pcs/box	40boxes/carton
6mm*75mm (3 Strips)	Wound of 50mm long	50pcs/box	40boxes/carton
6mm*38mm (6 Strips)	Wound of 50mm long	50pcs/box	40boxes/carton
3mm*75mm (5 Strips)	Wound of 50mm long	50pcs/box	40boxes/carton

Packing Details

Carton size: 35.5*33*17cm

G.W/N.W: 5.5/5kg

