



Strip



Cassette



Midstream

Hcg Pregnancy Test

Disposable

The Hcg Test is a rapid test device for the qualitative detection of human chorionic gonadotropin (HCG) in urine samples as an aid in the early detection of pregnancy.

Intended Use:

This test kit is intended for the qualitative detection of HCG in human urine samples as an aid in the early detection of pregnancy. It can be used for self-testing.

Material Composition:

- Monoclonal Anti-HCG Antibodies
- Colloidal Gold-Labeled Antibodies
- Membrane Carrier

All materials meet relevant regulatory standards.

Specifications:

- Item: Hcg Pregnancy Test
- Specimen: Urine
- Accuracy: 99%
- Sensitivity: 10-25mIU/ml
- Read time: within 5 minutes
- Shelf Life: 36 months





Specifications Details:

Item No	Item	Specimen	Sensitivity	Size(mm)	Photo
E20076S	Strip	Urine	10-25mIU/ml	2.5,3.0,4.0,5.0	<p>HCG TEST STRIP</p>
E20076C	Cassette	Urine	10-25mIU/ml	3.0,4.0	<p>HCG TEST CASSETTE</p>
E20076M	Midstream	Urine	10-25mIU/ml	3.0,5.5,6.0	<p>HCG PREGNANCY TEST</p>

Material Composition:

- High Sensitivity and Accuracy: Ensures reliable detection of HCG.
- Simple Operation: Easy to use with clear result interpretation.
- Compact and Portable: Suitable for self-testing at home.
- Hygienic and Safe: Designed for single-use to ensure safety.





Instructions for Use:


1. Remove the test cassette from the pouch by tearing the foil pouch.
2. Draw specimen (urine) into the dropper and dispense it into the sample well on the cassette (2-3 drops, approximately 100ul). Be careful not to overfill the absorbent pad.
3. Read results within five minutes.
4. Discard the test device after a single-use.
5. Depending on the concentration of HCG in the urine sample, positive results may be seen within 40 seconds. Wait the full five minutes to confirm the result. Do not read the Cassette Test after 5 minutes as this may also give an incorrect test result.

Results

Negative: Only one colored band appears in the control region. No colored band in the test region.

Positive: In addition to the control band, a distinct colored band also appears in the test region.

Invalid: If no bands appear, or a test band appears without a control band, the test should be repeated using a new test cassette.

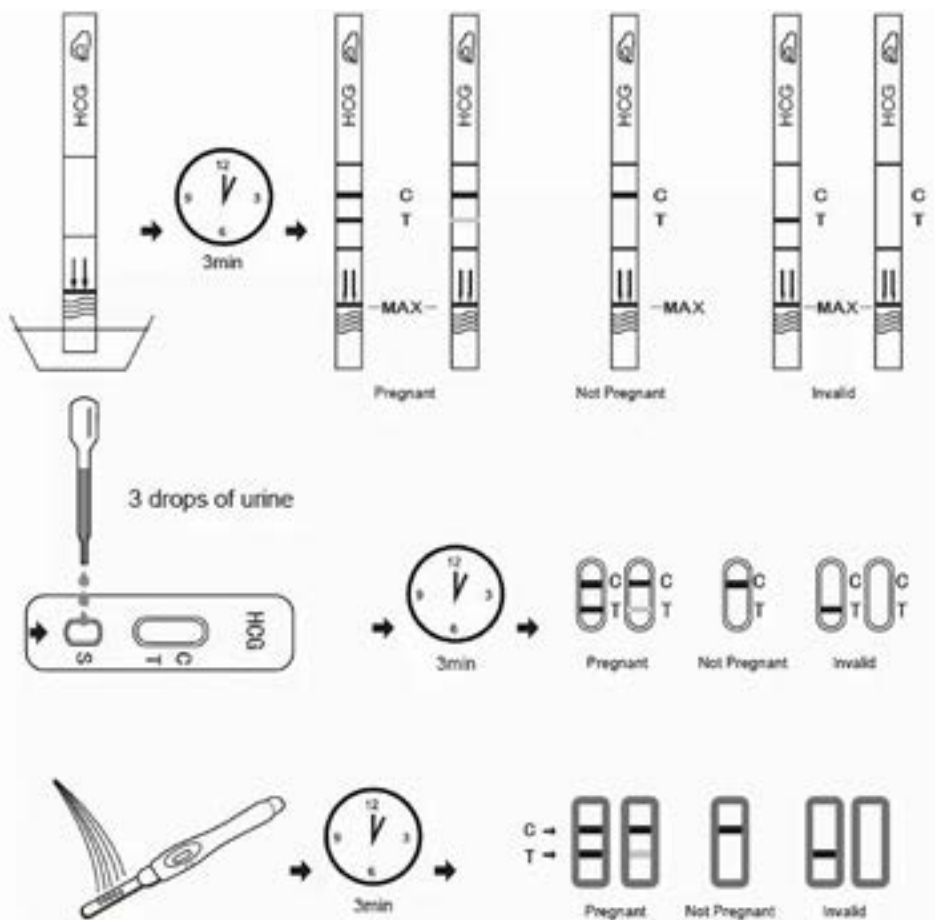



Storage Conditions:

Store the unopened product in a cool, dry place at a temperature between 2-30°C (36-86°F). Avoid direct sunlight.

Precautions:

- For in vitro diagnostic use only.
- Do not reuse the test device.
- Do not use beyond the expiration date.
- Follow the instructions for use strictly.
- If the result is positive, seek medical advice for further confirmation.

Packing Details:

Item No	Item	Packing	Carton size(cm)
E20076S	Strip	100pcs/box, 5000pcs/ctn	75*29*56
E20076C	Cassette	50pcs/box, 2000pcs/ctn	63*40*33
E20076M	Midstream	25pcs/box, 1000pcs/ctn 1 foli bag/box, 10 boxes/shrink, 500 pcs/ctn	63*46*34

