



Urine Bag (with Push Pull Valve)

Disposable

The urine bag with a push-pull valve is a high-quality medical device designed to collect and store urine for patients who require assistance with urinary management. Made from durable medical-grade materials, it ensures safety and comfort for users. The push-pull valve design facilitates easy control of urine discharge, enhancing user convenience and hygiene.

Intended Use

The urine bag is intended for use in hospitals, clinics, and home care settings to manage urine collection for patients who are bedridden, immobile, or require long-term urinary drainage. It is suitable for use with standard urinary catheters and is designed for single-use to prevent infection and maintain hygiene.

Material Composition

- Bag Material: Medical-grade PVC (Polyvinyl Chloride)
- Valve Material: PP (Polypropylene)
- Tubing Material: PVC or Silicone
- Hooks and Clamps: ABS (Acrylonitrile Butadiene Styrene)

Specifications

- | | |
|------------------------|--|
| ·Item: | Urine bag |
| ·Capacity: | 2000 ml |
| ·Bag Dimensions: | 27.5 x 19 cm |
| ·Tubing Length: | 90 cm |
| ·Valve Type: | Push-Pull Valve |
| ·Connector Diameter: | Standard connector fits most catheters |
| ·Sterilization Method: | Ethylene Oxide Sterilization |





Features and Benefits

- Ease of Use: Push-pull valve design allows for one-handed operation.
- Anti-Backflow: Built-in anti-backflow valve prevents urine from flowing back.
- Clear Scale: Transparent bag with graduated scale for easy urine volume monitoring.
- Durability: High-quality materials and leak-proof design ensure long service life.
- Comfort: Soft tubing and bag reduce skin irritation.

Instructions for Use

- Connection: Connect the urine bag's connector to the catheter securely.
- Fixation: Use the hooks to secure the urine bag to the bedside or other suitable locations.
- Discharge: When needing to empty the urine bag, pull out the push-pull valve's plunger, discharge urine, then push the plunger back to close the valve.
- Replacement: Regularly replace the urine bag to maintain hygiene.

Storage Conditions

- Temperature: Store in a cool, dry place away from direct sunlight.
- Humidity: Relative humidity not exceeding 80%.
- Other: Keep away from corrosive chemicals.

Precautions

- Single Use: This product is for single use only. Dispose of after use according to medical waste disposal regulations.
- Allergic Reactions: If an allergic reaction occurs, discontinue use immediately and consult a doctor.
- Keep Out of Reach of Children: Store in a place inaccessible to children.

Quality Certifications

- This product complies with ISO 13485 Medical Devices Quality Management System Certification
- CE certified, meeting EU Medical Device Directive.

Packing Details

Package:	250pcs/ctn
Carton size:	44x38x33cm
G.W/N.W:	13/12kgs

